



Vital Signs Monitor Reference Card

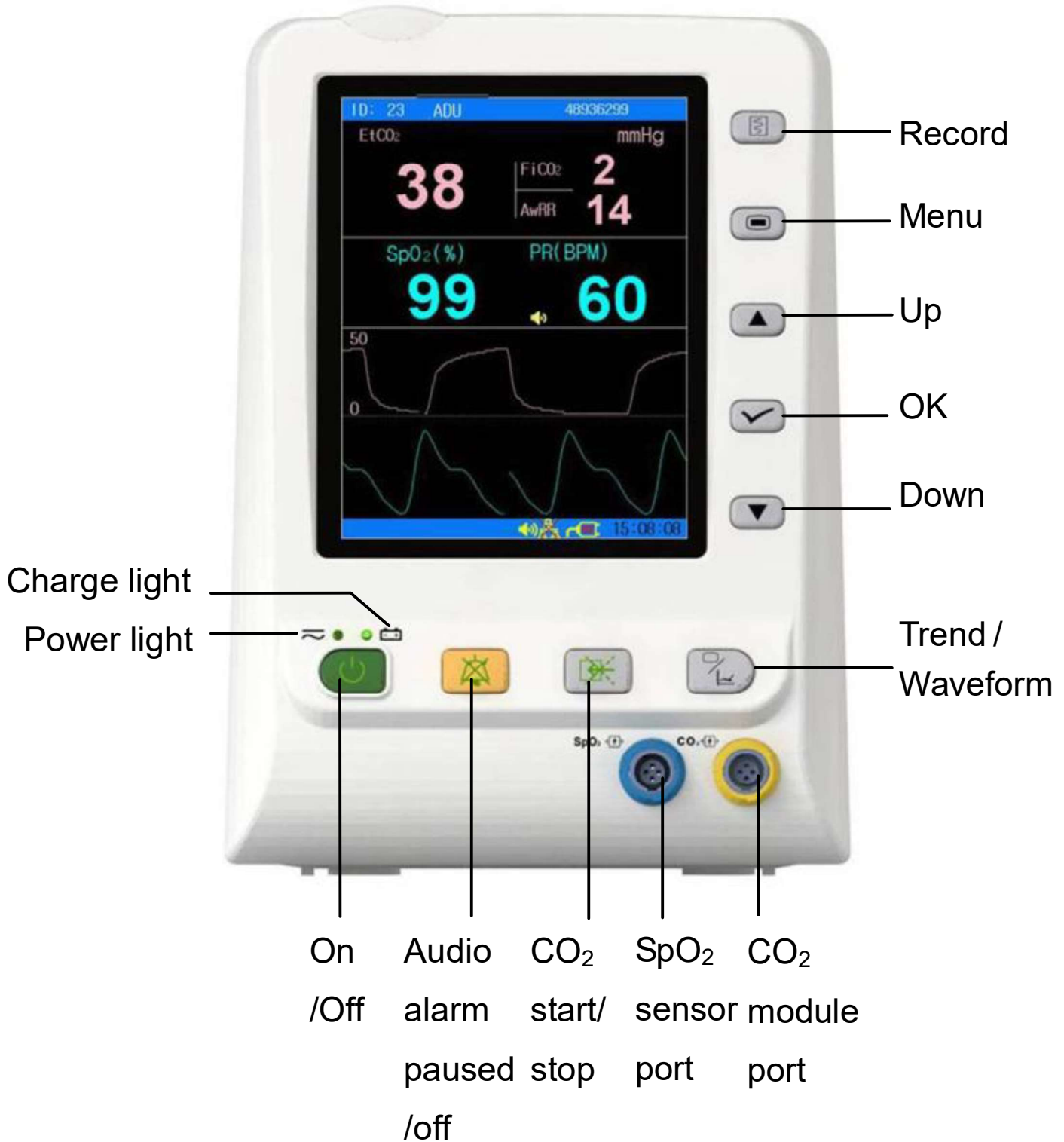

After being connected to various sensor connectors and powered on, the device can be operated normally.

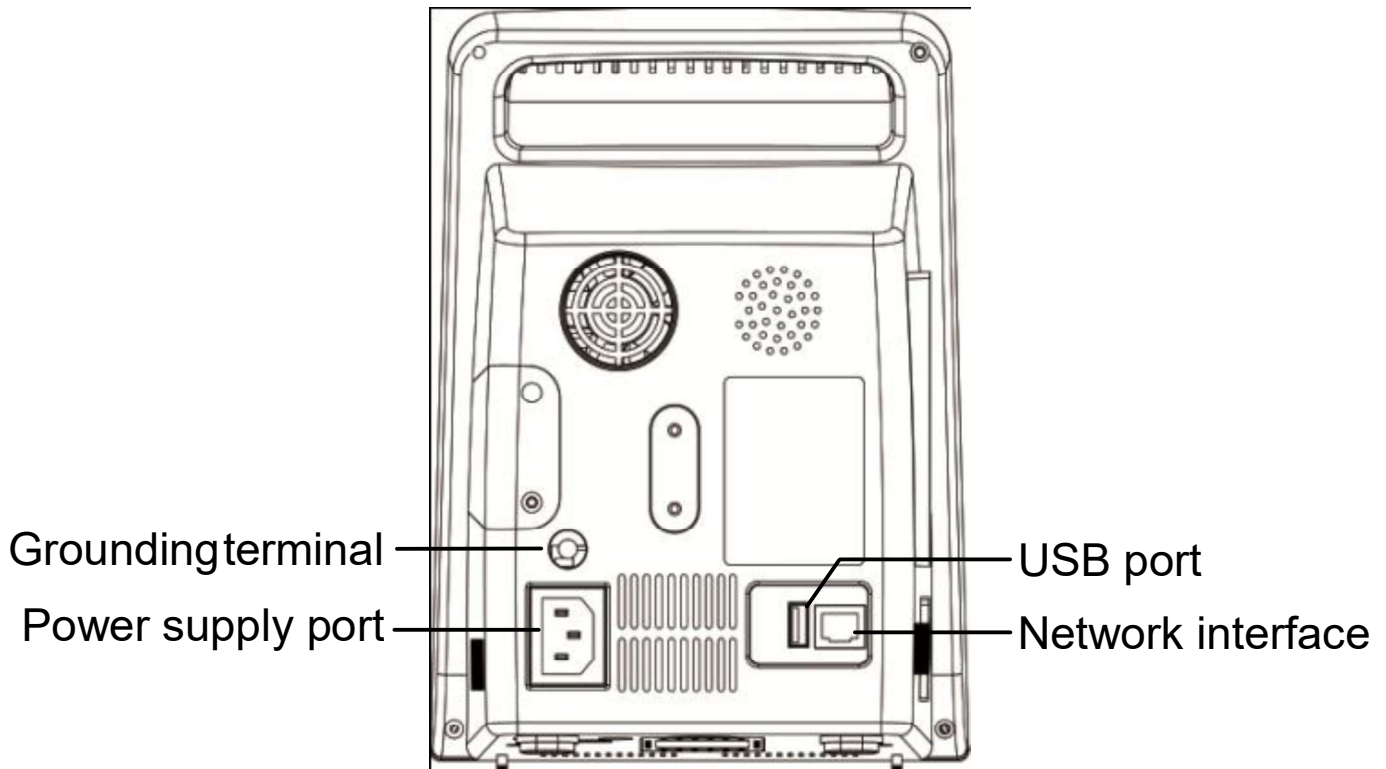

Front Panel Introduction



ON/OFF:

Press this button to power on the monitor. When the monitor is on, press $\geq 2s$ to turn it off, or $< 1s$ to enter Sleep mode.



AUDIO ALARM PAUSED/OFF:

Upon the configuration, pressing this button to pause or turn off the audio alarm. Further information can be found in the user manual Section *Audio Alarm Paused* and Section *Audio Alarm Off*.



CO₂ START/STOP:

Press to start CO₂ measurement. During the measuring process, press the button to stop measurement.



TREND/WAVEFORM:

Press this button to switch among waveform display, trend graph display and trend list display.



RECORD:

Press this button to print out the currently displayed contents.



MENU:

Press to call up **SYSTEM MENU**.



UP









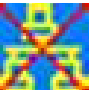





OK

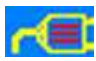




DOWN:

Press **UP/ DOWN** to select the items in menu or adjust its value.
Confirm by **OK**.

The icons on main interface:

	Power supply		High alarm
	Volume indicator		Medium/low alarm
	Network connection		Audio alarm off
	Network connection off		Audio alarm paused
	Battery status		Parameter alarm off
	Patient type: ADU		Heart beat

Vital Signs Monitor Reference Card

	Patient type: PED	ID	Current patient ID
	Patient type: NEO	15: 08: 08	Current time
	Barcode		

NOTE:

- 1 Before measuring, please check the patient type. Incorrect type may lead to inaccurate measurement or serious injuries. Set it in **SYSTEM MENU > PATIENT SETUP**.
- 2 The device must be connected to ground effectively to avoid interference signals.
- 3 During SpO₂ measuring, tissue damage may be caused by incorrect operation or prolonged measuring duration (more than 2 hours).
- 4 Be careful when inserting paper to recorder. Avoid damaging the thermo-sensitive print head. Unless when inserting paper or shooting troubles, do not leave the recorder casing open.
- 5 If you do not use the equipment for more than one month, please take out the battery, and charge it once every month.
- 6 If the alarm is turned off, the monitor can not give an alarm prompt when an alarm occurs, so the user should use this function cautiously.

Note: This quick reference card is not intended as a substitute for the user manual. The user must be familiar with the information in the user manual before monitoring patients.

P/N: 01.54.109453